



MEDintim
personal healthcare



Oralsafe

Latex sheet for oral-genital contact

The new **Oralsafe** latex sheets are the best protection for oral-genital contact.

Instruction to use:

Open carefully the foiled latex tissue. Moisture the vaginal or anal part of the body with a water-based lubricant. Place well the **Oralsafe** tissue to cover the vagina with clitoris or anus. Fix the tissue at its place while you have oral-genital contact. Dispose the used tissue at your waste. Don't flush into the toilette.

Notice: Use always a new **Oralsafe** sheet when you switch to use it from anal to vaginal areas. Do not cut or split up one sheet. Please use one sheet only one time. If you or your partner uses vaginally pharmaceuticals, please check with your healthcare provider if this pharmaceutical is compatible to latex. If you have latex-allergy, do not use this medical device – it may cause allergic reactions. **Oralsafe** protects you and your partner against sexual transmitted infections (STI). No method can protect 100% against STIs.

Ingrédients:

natural latex rubber

Contents:

8 x 1 piece latex tissue for single use,
available with strawberry or vanilla aroma
size: 252 x 115 mm
colour: beige
box: 65 x 93 x 19 mm

Storage:

Keep dry, cool and out of direct sunlight (UV)

Manufacturer:

Kessel Marketing & Vertriebs GmbH

Strawberry EAN 4013273900164

Vanilla EAN 4013273900171



Medical Device



Kessel Marketing & Vertriebs GmbH
Kelsterbacher Straße 28
64546 Mörfelden-Walldorf, Germany

Tel. +49 (0) 6105 20 37 20
Fax +49 (0) 6105 45 59 01
E-Mail: service@medintim.de