

Medintim



Manual
Erection System

Vakuumtherapie bei Erektile Dysfunktion (ED)

Medintim

Aufsetzen des
Pumpenkopfes.

Manual
Erection System

Medintim

Aufsetzen der
Führungshülse II.

Manual
Erection System



Medintim

Aufsetzen der
Führungshülse I.

Manual
Erection System



Medintim

Aufsetzen
des Konus.

Manual
Erection System



Medintim

Aufschieben eines
Stauringes
mit dem Konus.

Manual
Erection System



Medintim

**Alternativ:
Aufspannen
des Stauringes
mit den Händen.**

Manual
Erection System





Medintim

Anwendung am Körper



Medintim

Auftragen
von Gleitgel.

Manual
Erection System



Medintim

Einführen des
Penis in den
Zylinder.

Manual
Erection System

Medintim

Heranführen
an den Körper
und pumpen.



Manual
Erection System

Medintim

Stauring auf
den Penis
schieben.

Manual
Erection System



Medintim

Erektion, durch Stauring
gehalten (max. 30 Min.)

Manual
Erection System



Medintim

**Nach der Anwendung:
Stauring auseinander ziehen
und entfernen.**

Manual
Erection System

A photograph showing a man's hands holding a clear plastic ring and a small black object. A callout line points from the text 'Zustand nach der Anwendung.' to the man's penis, which is shown in an erect state. The background is a warm, orange-brown gradient.

Medintim

Zustand nach der Anwendung.

Manual
Erection System

Medintim



FDA registered device
Made in Germany



KESSEL Medintim GmbH
Kelsterbacher Str. 28
64546 Mörfelden-Walldorf
Germany
+ 49 (0) 6105/203 720

www.medintim.de