



Material:

100 % TPE (thermoplastic elastomer)

EAN 4013273000857

Made in Germany

Manufacturer:

KESSEL medintim GmbH

Nordendstr. 82 – 84

64546 Mörfelden-Walldorf

Germany

e-mail: export@medintim.de

www.medintim.de

Backup

for improved, longer lasting
erections

What is the Backup Ring?

The backup ring helps men with mild erectile dysfunction. It improves the firmness and fullness of the penis. One size fits all.

The Backup Ring helps men who are not satisfied with the quality of their erection or whose erection is not firm enough for the sexual activity. When the erection is improved, both partners can have a more intense feeling.




Note for men using a vacuum erection device with a constriction ring together with the Backup ring:


First use the vacuum erection device with the constriction ring as normal. As soon as the constriction ring is in place at the root of the penis, take the Backup ring and pull it apart. Next, slip the Backup ring over the testicles and the penis.

Application:

Simply pull apart (fig. 1), slip over the erect penis (fig. 2) and place behind the testicles (fig. 3).

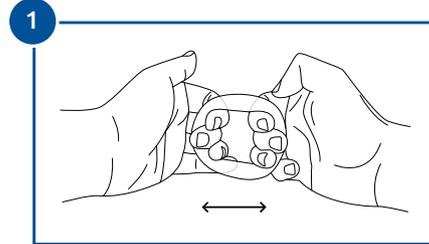

The backup ring can be slipped over the penis and testicles at any time, regardless of the arousal of the penis.

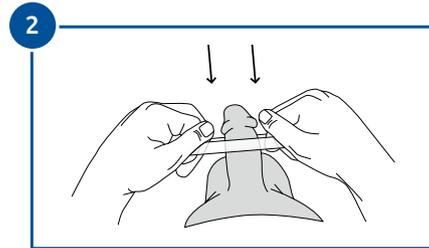

The Backup ring sits between the testicles and the body at the root of the penis. It presses against the base of the penis. This ring guides and stabilises the penis and testicles.

At the top, where the shaft meets the pubic bone, the ring can have a stimulating effect on the female partner.

