Anello Backup

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- 1 Backup Ring
- Gebrauchsanweisung

Backup Ring

makes erections harder and longer-lasting

The Backup ring improves the firmness and fullness of the penis. One size fits all.

How to use:

Simply pull apart, slip over the erect penis and place behind the testicles.

The Backup ring sits between the testicles and the body at the root of the penis. It presses against the base of the penis. This ring guides and stabilises the penis and testicles.

At the top, where the shaft meets the pubic bone, the ring can have a stimulating effect on the female partner.

Important: The Backup ring is always placed behind the testicles.

The Backup ring makes erections harder and better – they last longer. Both partners have a more intense feeling.

Material:

TPE (thermoplastic elastomer)

Note for men using a vacuum erection aid plus retaining ring together with the Backup ring:

First use the vacuum erection aid plus constriction ring as normal. As soon as the constriction ring is in place at the root of the penis, take the Backup ring and pull it apart. Next, slip the Backup ring over the testicles and the penis.

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