Helpi Ring

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Helpi Ring

What is the Helpi® Ring?

The Helpi® Ring keeps your erections hard and prevents premature, visible ejaculation.*

A simple solution without drugs, creams or gels.

How to use:

Stretch the ring onto the blue sleeve (Fig. 1).

Push the sleeve, together with the ring, over the penis towards your body (Fig. 2).

Slip the ring off the sleeve and onto the penis (Fig. 3).

Now remove the sleeve (Fig. 4).

The Helpi® Ring will now maintain the erection regardless of when the user comes. The erection is maintained even after the man climaxes. He can continue to have sex. The Helpi® Ring does not interfere and can even have a stimulating effect on the partner (Fig. 5).

The user can easily remove the Helpi® Ring at any time. Pull on the flaps and wait a few seconds until the erection subsides. You can now simply slip the ring off (Fig. 6).

Note:

Do not leave the ring on the penis for more than 30 minutes at a time. We recommend removing hair from the shaft beforehand.

Use and care:

All parts are re-usable. The rings are subject to wear and tear. They may split after frequent use. If this happens new rings must be ordered.

Clean with warm water and soap. Store in the pouch provided.

The Helpi® Ring is a safe and easy way of removing stress.

*Premature ejaculation (ejaculatio praecox) may be a symptom of a condition requiring treatment, such as prostatitis or hyperthyroidism. So it's important to consult a doctor as soon as possible.

Contents per pack:

2 x Helpi® Rings made of TPE
1 x polycarbonate sleeve (blue)
1 x Instructions For Use
1 x cotton pouch for storage

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