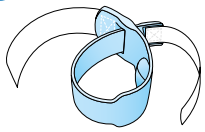
1

## Prosecca® Band

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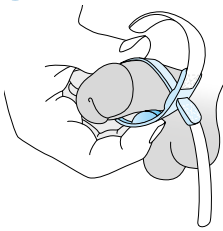
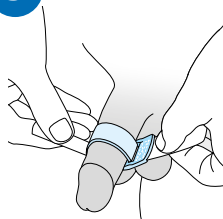
## Prosecca® Band

A band to prevent leaking

By exerting pressure on the urethra, the Prosecca® Band stops urine leaks.

### Indication:

- The Prosecca® Band helps men with mild to moderate bladder weakness with post urinary dribbling. This can be caused by:
- narrowed urethra
  - (benign) enlargement of the prostate
  - consequences of an operation on the bladder
  - late sequelae of radiation
  - inactivity of the muscles and nerves around the urethra
  - decreased blood flow

### Contraindication:

The Prosecca® Band should not be used in the event of acute urinary tract infections or severe incontinence.

### Properties:

- smooth, supple material
- practical and reusable
- Velcro® fastener for easy adjustment of the band to the desired pressure on the urethra
- light and narrow in size
- comfortable to wear whilst running, sitting, walking or doing sports
- smooth surface, easy to clean
- with practical storage box

### Application:

Open the Prosecca® Band until it is wide enough for you to slip over your penis. The pressure cushion should sit in the middle of the penis on the underside. Sitting in the right position, pull the ends of the Velcro® strap with both hands. As tight as possible, but not too tight that it is painful. Then close the Velcro® fastener. Look to see if the leaking has stopped.

### Note:

If you see any dark red or bluish discoloration, the Prosecca® Band is exerting too much pressure and needs to be loosened a little. Open the band at least every 2 hours or remove it and empty your bladder.

The Prosecca® Band is for personal use only. It may not be passed on to others.

Remove the Prosecca® Band before going to the toilet!

### Cleaning:

The Prosecca® Band should be cleaned with warm water and soap before you use it for the first time and after every time you wear it. To protect the closure, the Prosecca® Band should only be washed when closed. Dry it before you put it back in the box provided. The Prosecca® Band is reusable.

### When does the Prosecca® Band need to be replaced?

- If you notice that the band has torn.
- If you notice that the pressure cushion is severely compressed.
- If the Velcro® fastener is loose or closes poorly.
- When the band can no longer be held in position.

### Material:

Polyamide, Polyester, Polypropylene, Thermoplastic Elastomer (TPE)

### Note:

The product should not be used in the event of known intolerance to the material mentioned.

Serious incidents as well as unknown serious adverse reactions must be reported immediately to the manufacturer and the competent authority.

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KESSEL medintim GmbH