# Animé Lubricant

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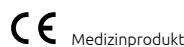
# Animé Lubricant

**Animé lubricant with hyaluronic acid**  
Intimate lubricant for her and him

## Instruction manual

Read the instructions for use before using.

## Composition

Purified water, propylene glycol, glycerol, hydroxyethyl cellulose, benzoic acid, sodium hyaluronate

## Properties

- waterbased
- free of silicones and parabens
- colourless
- odourless
- pH Value 3.5 - 4.5
- contains hyaluron

Hyaluron can support the healing of the skin and tears.

## Indications

Gel to improve lubrication during sexual intercourse. In case of vaginal dryness. Can also be used on condoms and for inserting tampons or thermometers and aids for sexual stimulation.

## Contraindications

Known hypersensitivity to one of the ingredients. In cases of vaginal infection, ask your doctor before using the product.

## Use

Apply the amount individually needed directly to the skin or apply to the desired site with a finger.

This product is compatible with condoms made of natural rubber latex, polyurethane and polyisoprene. It has **no** contraceptive effect.

## Note

If the packaging is damaged such that the tube is no longer intact, do not use the product. Keep medical device out of the reach of children.

## Side effects

There are no reports of side effects to date.

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KESSEL medintim GmbH