Contragel®

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Contragel®

Instructions for use

(Please read carefully and keep for future reference.)

Contragel® green is a contraceptive gel for women to use with diaphragms as a method of barrier contraception.

Application:

Before inserting into the vagina, place about one teaspoonful (about 4 ml) of Contragel® green in the diaphragm. Then follow the instructions for inserting the diaphragm.

The cellulose gel creates an additional mechanical barrier between the diaphragm and the uterine orifice.

The lactic acid with its physiological pH of 3.8 has a motility inhibiting effect on the sperms - they are slowed in their forward movement or stopped.

Composition:

Water, lactic acid, sodium lactate, cellulose, sorbic acid, vanillin.

Storage:

Store in a dry place between 4°C and 25°C. Always keep the tube closed. For minimum shelf-life, see printing on tube.

Tubes that are not completely empty may also be disposed of with household waste.

Side effects:

Contragel® green because of its pH, is well tolerated by the vaginal flora. It can be used for as long and as often as required. If you notice any adverse effects during use, stop using Contragel® green and ask your doctor before further use.

Interactions:
None known.

Contraindications:

There have been no reports of women experiencing side effects during the use of Contragel® green. Contragel® green should not be used if you are known to be hypersensitive or have an allergy to one of the ingredients. If you experience severe burning or itching during use, stop using the gel and contact your doctor.

Important:

If any serious incidents occur whilst using Contragel® green, notify the manufacturer and competent authority.

Important:

Contragel® green offers NO protection against sexually transmitted diseases. It is only effective when used together with a diaphragm (e.g. Caya® contoured diaphragm) cap. Contragel® green IS NOT a safe contraceptive method when used alone without a mechanical barrier. Contragel® green is water-soluble and compatible with condoms made of natural rubber latex or polyurethane.

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KESSEL medintim GmbH
DeltaMed GmbH