

Prefert® Vaginal Gel for couples trying to conceive

Prefert® Vaginal Gel is a lubricant that supports sperm activity. Prefert® Vaginal Gels available in pharmacies or online without prescription.



Ingredients:

Purified Water, Sodium Chloride, Sodium Phosphate, Potassium Phosphate, Arabinogalactan, Hydroxyethylcellulose, Polyhexanide

Free of parabens, silicones, mineral oils and animal products, vegan.

Content:

- 8 pre-filled single dose applicators Prefert® Vaginal Gel with 4 ml gel each.

The content of this package is good for 4 – 8 ovulation cycles.

After use:

Discard the empty syringes in household waste.



Medical Device
Made in Germany

Manufacturer:

KESSEL medintim GmbH
Nordendstr. 82 – 84
64546 Mörfelden-Walldorf
Germany

Phone: +49 6105 20 37 20

www.medintim.de



Prefert®

Vaginal Gel

Lubricant when trying to conceive

Medintim

Prefert®

Vaginal Gel

A lubricant that supports your natural fertility and helps getting pregnant.

Every seventh couple's wish for a baby remains unfulfilled. One possible reason: a too acidic pH value during a woman's fertile days (ovulation).

Prefert® Vaginal Gel is unique and supports a sperm friendly environment. Survival and mobility of the sperm cells is improved and therefore increases the chance to conceive.

Many couples put themselves under pressure and have sex only at the right time. Making love exactly at the time of ovulation can also be stressful. This can lead to a dry vagina, which can cause pain during sex.

Prefert® Vaginal Gel makes it easier and more pleasurable. Conventional, water-based lubricants have a low pH value (=acidic) and have a negative effect on sperm cells. These lubricants reduce the chance to conceive and change the viscosity of the vaginal secretion.



Free of silicones, parabens & mineral oils



Optimized conditions to conceive

Prefert® Vaginal Gel creates the perfect vaginal pH value in the vagina to conceive. In this way, it supports sperm to reach the egg.

Application:

Before having sex, Prefert Vaginal Gel is inserted directly into the vagina with the single dose applicator. The gel moisturizes and creates the perfect environment for sperm cells. Lubrication makes sex more pleasurable and less painful while trying to conceive.

Tip:

You can check if your vaginal pH is too acidic at the time of ovulation. Get yourself a pH test stick (pharmacy) and use it to measure the pH value at the time of ovulation. If the value is 5.0 or below, the environment is too acidic to get pregnant. Prefert® Vaginal Gel is the right lubricant to help you.

Prefert® Vaginal Gel with its adjusted pH value of 7.0 – 7.4 is perfect for sperm cells. This is like the natural pH value of the vagina during the fertile days. On the other days of the female cycle, the pH value is lower and protects from infections. The osmotic properties of Prefert® Vaginal Gel help the sperm reach the cervix more easily.

Properties:

- free of silicones and parabens
- odor- and colorless
- tasteless
- makes no spots
- water-soluble
- vegan

Note:

If you have had recently any vaginal surgery, ask your healthcare provider before use. Do not use Prefert® Vaginal Gel, if you have an active vaginal infection.