

**Attention:**

Do not use in case of acute urinary tract infections or severe incontinence.



**Material Velcro® straps:** Velour tape 100 % polyamide, Mushroom-head tape: 70 % polyester, 30 % polypropylene

**Material blue tourniquet:** 100 % TPE

**Length:** approx. 280 mm / **Width:** 25 mm

**Diameter:** adjustable

**EAN** 4013273001908

Made in Germany

**Manufacturer:**

**KESSEL medintim GmbH**

Nordendstr. 82 – 84

64546 Mörfelden-Walldorf

Germany

[www.medintim.de](http://www.medintim.de)

New!



# Prosecca®

A band to prevent  
leaking

05.03.2021

Medintim

Many men suffer from incontinence. Even slight dripping has a strong impact on the quality of life. Therefore, the Prosecca® band is an effective solution. With pressure on the urethra, it stops leak of urine. Its special material makes it particularly comfortable and discreet to wear.

## Advantages of the Prosecca® band:

- smooth, supple material
- practical and reusable
- Velcro® fastener to adjust the diameter and pressure on the urethra
- light and narrow in size
- discreet and unobtrusive to wear
- comfortable to wear whilst running, sitting, walking or doing sports
- smooth surface, easy to clean
- with practical storage box

## How to use the Prosecca® band:

The Prosecca® band is opened until it is wide enough to slip over the penis. The pressure cushion should be placed in the middle underneath the penis. Sitting in the right position, the band is tightened. As tight as possible, but not too tight that it is painful. Now the Velcro® fastener is closed and the leaking is stopped.

## Reusable and easy to clean:

The Prosecca® band is reusable. If used properly and with care it will last a long time and can be used many times. It should be cleaned with warm water and soap before first use and after every following use. When dried, it may be put back in the box provided.

