Gebrauchsanweisung beachten /
Read instructions for use / Consulter la notice

Medizinprodukt / Medical device /
Dispositif médical

Hersteller / Manufacturer / Fabricant

Vertrieb / Distributor / Distribution

KESSEL medintim GmbH
Nordendstr. 82 – 84
64546 Mörfelden-Walldorf
Germany

T: +49 6105 20 37 20
www.medintim.de

HdB
Laagstraat 59
9140 Temse
Belgium

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Vaginal applicator for gel or cream

Instruction for use:

Thoroughly wash your hands.

1. Fill the applicator with the gel or cream. Screw the applicator onto the tube and carefully squeeze the gel or cream out of the tube until the indicated dosage mark is reached.
2. Remove the applicator from the tube.
3. Insert the applicator as deeply as possible into the vagina. This is most simply performed lying down with the legs slightly raised. In this position, the pelvic muscles are relaxed.
4. Once you have correctly positioned the applicator, empty it by gently pressing down on the plunger.
5. Remove the empty applicator from the vagina without pulling on the plunger, as some of the gel or cream would otherwise be sucked back into the applicator.
6. Once used, dispose of single-use applicators in the household waste.

Important information:

- Do not use gel or cream applicators during pregnancy. There is a risk of injury if handled incorrectly, as the area around the cervix is very sensitive in pregnant women. Ask your doctor for more information.
- Do not administer during menstruation. Talk to your doctor if you are shortly due to have a period at the start of treatment.
- The applicator fits tubes with 9 mm thread, e. g. Caya® Gel or Contragel® green.

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