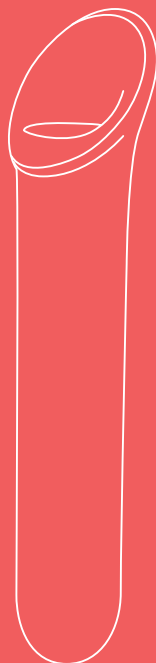
Vagiwell®

Dilators

1	DE	Vaginale Dilatatoren aus Silikon	
5	EN	Silicone vaginal dilators	
9	FR	Dilatateurs vaginaux en silicone	
13	ES	Dilatadores vaginales de silicona	
17	IT	Dilatatori vaginali in silicone	
21	EL	Κολπικοί διαστολείς από σιλίκονη	
25	HU	Szilikonból készült hüvelyi dilatátorok	
29	DA	Vaginale dilatatorer af silikone	
33	NL	Vaginale dilatatoren van siliconen	
37	NO	Vaginale dilatorer av silikon	
41	FI	Silikoniset emättimen laajentimet	
45	SV	Vaginala dilatatorer av silikon	
49	PL	Rozszerzacz pochwy z silikonu	
53	CZ	Silikonové vaginální dilatátory	
57	SK	Vaginálne dilatátory zo silikónu	
61	MS	Dilator faraj diperbuat daripada silikon	
65	HR	Vaginalni dilatatori izrađeni od silikona	
69	PT	Dilatadores vaginais de silicone	

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Vagiwell® instructions for use

Read these instructions for use carefully, keep them for later reference, follow the instructions.

Items supplied:

Check the cardboard packaging for external damage and ensure that the contents are complete. Before use, it should be ensured that the dilators do not have any visible damage and that any packaging material is removed. If in doubt, do not use the dilators and contact the manufacturer.

Also included are:

- Lubricant (CE-certified)*
- Storage bag*

*Not included in the case of Vagiwell® size 0.

Important: Please note the shelf life of the lubricant provided.

Intended purpose:

The use of vaginal dilators represents a therapeutic option for increasing familiarity with one's own body and enabling painless intravaginal sexual intercourse and gynaecological examinations as well as preserving the ability to engage in intercourse following radiation therapy/chemotherapy and enabling dilation following sex reassignment surgery, among other things.

The aim of the exercises is desensitisation in which the vagina is to be slowly stretched so that (once again) painless sexual intercourse or gynaecological examinations are possible.

Target group:

The dilators are intended for personal use in a home environment, not in professional healthcare facilities. The application is suitable for all adult persons.

Important: The dilators are to be used by one person only. Passing them on to third parties does not comply with the intended purpose.

Indications:

- Dyspareunia (recurring or constant genital pain during sexual intercourse) of different causes, e.g.:
 - vaginitis (inflammation of the vagina)
 - adnexitis (combined inflammation of the fallopian tubes, ovaries and surrounding tissue)
 - vaginal stenosis
 - endometriosis
- vaginismus (tightening of the vaginal muscles)
- scarring following surgical cuts to the perineum (episiotomy) or perineal tears
- subsequent to gynaecological operations (e.g. prolapse surgery or cancer operations)
- during and after radiotherapy/chemotherapy
- after vaginal surgery involving tightening of the vagina and scarring
- skin conditions such as certain lichen forms, e.g. lichen sclerosis
- vaginoplasty/neovagina (e.g. during sex reassignment surgery)
- narrowness of the vagina
- Vaginismus due to various causes, such as:
 - psychological causes, such as anxiety and phobias
 - physical causes (endometriosis, multiple sclerosis, diabetes, etc.)
- Vaginal stenosis due to various causes:
 - lichen sclerosis
 - childbirth-related injuries
 - prolapse surgery
 - vulvar carcinoma
 - cervical and uterine carcinoma
 - chronic graft vs. host reaction
 - radiation after-effects

Stretching exercises with the Vagiwell® dilators can also be useful in making gynaecological examinations or the insertion of tampons less painful.

Contraindications:

There are no reports of contraindications. The Vagiwell® dilators should not be used in the case of known hypersensitivity or allergy to the material (silicone). Dilators should not be used in the case of acute infections of the genital tract and inflammation of the abdomen. In the case of severe pain during use, the therapy should be discontinued and a doctor should be consulted.

Side effects:

There are no reports of side effects. In the case of hypersensitivity or allergic reactions to the material (silicone), the therapy should be discontinued and the doctor should be consulted.

Interactions:

None known.

Note: Please notify the manufacturer or the competent authority if serious incidents occur during the use of the Vagiwell® dilators. For questions regarding the product or to report side effects/incidents, please use the following contact information:

KESSEL medintim GmbH
Nordendstraße 82–84
64546 Mörfelden-Walldorf
Phone: 06105 – 20 37 20
Email: service@medintim.de

Application:

The aim of the exercises is to desensitise the vagina by slowly stretching it, thereby making sexual intercourse pain-free (once more) and gynaecological examinations a painless experience. Do the exercises in a place where you feel comfortable and are relaxed. Make sure that neither the telephone nor anything else will disturb you while you are carrying out the exercises.

1. Prior to use, wash the Vagiwell® dilators with hot water and a mild soap and dry them thoroughly.

2. Find a relaxed sitting or lying position that allows you to spread your legs slightly. For inserting the dilators, we recommend lying flat on your back with your knees bent and your legs spread slightly apart. You can, however, also use the dilators standing up, with one leg on a chair.
3. Lubricate the entrance to the vagina and the dilator with a water-soluble lubricant gel.
4. Breathe calmly. Gently push the dilator into the vagina. You should only insert the dilator as far in as is comfortable for you. Should this be painful, you may be using too much pressure.
5. Leave the dilator in the vagina or in the entrance to the vagina for a few minutes.
6. After that, insert and remove the dilator, without fully taking the dilator out of the vagina. Repeat this movement approx. 5 times. Then slowly start moving the dilator with a slight rotating movement: rotate the dilator 5 times to the right and 5 times to the left. These movements help to gently dilate the vaginal tissue.
7. When you want to finish the stretching exercise, remove the dilator from the vagina. Remain lying down and relax for a few minutes more.

Once you are able to insert one size comfortably, your vagina has stretched far enough for you to be able to use the next largest dilator. When changing from one size to the next one up, the most recently used dilator should first be inserted and left in the vagina for a few minutes. Proceed with the next largest dilator as described above. The stretching exercises may be repeated three to four times a week. Once you are able to comfortably insert the largest dilator, you can consider having sexual intercourse.

Application in cases of lichen sclerosus and vulvodynia:

For women with lichen sclerosus, vulvodynia or lichen planus, a different form of application is recommended. Before it is actually stretched using the Vagiwell® sticks, it is helpful to gently massage the vulva and prepare it for being stretched.

1. Select the dilator (size) that is appropriate for your personal situation and that you can easily insert in the vagina.
2. Insert the dilator (moistened with lubricant) a maximum of 1–2 cm into the vagina – not deeper.
3. Then begin the dilation by using the dilator to gently press the tissue from the inside outwards several times.
4. Using this stretching motion, move around the entire vaginal entrance in a circle.

Note: It is important to be very careful in order to avoid tearing or injuring the skin. Particular caution is advised at the front of the vaginal entrance in order to avoid injuring the urethral opening. It is advisable to initially avoid this area when performing dilation. Patience and daily dilation, if possible, are important until the vulvar entrance becomes more flexible and the tissue stretches. You should change to the next size of dilator only after this has become possible. It may take days or weeks before size 5 can also be inserted. Many women also use the even larger “5 LS” dilator later on. The daily use of this dilator can keep the vulvar entrance flexible.

In some countries, there are patient organisations for lichen sclerosus and vulvodynia where you can obtain information and get more tips from users (e.g. www.lichensclerosus.ch). We have taken these tips from the experts at www.lichensclerosus.ch.

Cleaning:

The Vagiwell® dilators are intended for single-patient use. After use, wash the dilator(s) with warm water and mild soap. Make sure that the dilator(s) is/are dry before putting it/them back in the storage pouch (sets only).

Disposal:

The dilators can be disposed of in household waste.

Material:

- Dilators: med. silicone
- Pouch: cotton

Vagiwell® Dilators variants:

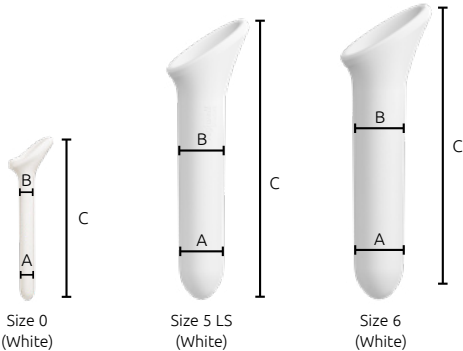
The Vagiwell® dilators are available in different variants (sets).

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