




ACTIVE *Erection System*^{NT}[®]

Vacuum erection aid
for erectile dysfunction

Indication:
erectile dysfunction caused by:

- Diabetes
- Hypertension (high blood pressure)
- Disorder of the cardiovascular system
- High blood lipid concentration
- Depressions or other mental disorders
- Neuropathic disorder (multiple sclerosis, paraplegia)
- Prostate gland disorder and/or prostate surgery
- Vascular disease of the glands penis
- Disorder of the liver or kidneys
- cardiac infarction or stroke
- side effects of pharmaceuticals (medication)
(for e.g. against hypertension, irregular heartbeat,
high blood lipid concentration, anti-inflammatory,
anti-depressive and for dehydration)
- hormonal or mental disorder
- alcohol and drug abuse
- smoking

The **ACTIVE Erection System**^{NT}[®] is an alternative
to treatment with drugs.

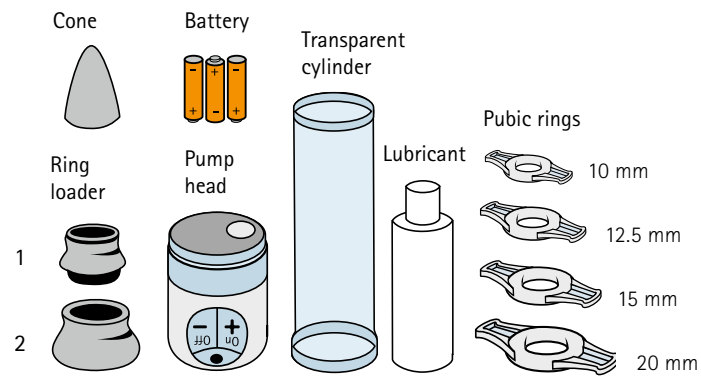


The **ACTIVE Erection System**^{NT}[®] consists of:

- Transparent cylinder
- Electric vacuum pump with three AAA batteries
- Ring loader 1 and 2
- Cone
- Four different pubic rings
(internal diameter 10, 12.5, 15 and 20 mm)
- Animé lubricant (50 ml bottle)
- Instructions for use + DVD
- Bag for storage

EAN 4013273000116

Art. No.: AES NT

Made in Germany

 0483 Medical device



KESSEL medintim GmbH
Nordendstr. 82 - 84
64546 Mörfelden-Walldorf, Germany

Tel. +49 6105 20 37 20
Fax +49 6105 20 37 221
Email: service@medintim.de



AES NT 00017



ACTIVE *Erection System*^{NT}[®]

Vacuum erection aid
for erectile dysfunction

www.medintim.de


MEDintim
personal healthcare



ACTIVE *Erection System*^{NT}®

Vacuum erection aid
for erectile dysfunction

Application

The **ACTIVE *Erection System*^{NT}**® is a medical aid to achieving an erection.

The vacuum pump creates a vacuum in the cylinder with the penis inserted. This causes the erectile tissue to swell and the penis become erect. One of the pubic rings is then slipped over the shaft of the penis. This slows down the blood flow when the cylinder is removed, and thus the erection is maintained. In this way normal and satisfying intercourse is possible.

All components are re-usable and are stored in the bag supplied.



Features

The **ACTIVE *Erection System*^{NT}**® is easily cleaned using warm water (apart from the pump). All parts, including the different sizes of pubic rings, can be re-ordered separately if required, even years later.

There is a 2 year warranty on the pump (motor).

The **ACTIVE *Erection System*^{NT}**® is an aid recognized by German health insurance companies. Talk to your doctor about this device.

Further information can be found at:
www.medintim.de