

## Medical vacuum erectile aids

### ACTIVE *Erection System*<sup>NT</sup>



### MANUAL *Erection System*



## ACTIVE *Erection System*<sup>NT</sup>

consists of the following parts:

- Cone
- Guide sleeves
- Cylinder, transparent
- Pump head, electrical, with three 1.5-volt AAA batteries
- 6 different constriction rings in sizes:  
8, 10, 12.5, 15, 20 and 22 mm
- Lubricant
- Instructions for use
- Storage pouch
- DVD with animation film (instructions for use)

### Product information

The **ACTIVE *Erection System*<sup>NT</sup>** and the manual **MANUAL *Erection System*** are medical aids to help achieve an erection.

### How does a vacuum pump work?

A vacuum pump generates a vacuum in the cylinder around the penis. This causes the erectile tissue to fill with blood and leads to stiffening of the penis. This erection can be maintained by placing one of the constriction rings provided over the penile shaft down to the base of the penis.

## MANUAL *Erection System*

consists of the following parts:

- Cone
- Guide sleeves
- Cylinder, transparent
- Pump head with lever
- 6 different constriction rings in sizes:  
8, 10, 12.5, 15, 20 und 22 mm
- Lubricant
- Instructions for use
- Storage pouch
- DVD with animation film (instructions for use)

The constriction prevents the return of blood and in this way maintains the erection. With the erection maintained in this way, sex and normal, satisfying intercourse are possible (for up to 30 minutes). Remove the constriction ring after sex. All parts of the aid are stored in the supplied pouch until reuse.

### Use


Both erection aids are easy to use. If you have practical questions, we would also be happy to provide you with help by telephone.

Both device variants come with a two-year guarantee. We manufacture in Germany and provide quality and service. Replacement parts or consumable parts can be ordered from us directly and easily.

Later on, when both partners are familiar with the vacuum erection aid, the chances for both of them to once again experience a satisfying sex life are good, even in older age or in cases of illness or physical limitations.

In our experience, it has been shown to help 95% of all men. The method is purely physical and does not affect the body chemically (systemically). Taking medications is not necessary. The vacuum pump helps many men, particularly those for whom the other methods do not work or are not considered.

Your local partner and distributor:

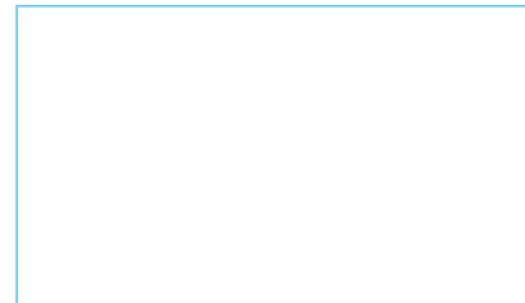

### Sales:

All of our aids are available in pharmacies, health care supply stores or through direct mail order.



## Has my partner lost desire for sex?

### What every woman should know about a partner's erectile problems.



KESSEL medintim GmbH  
Kelsterbacher Straße 28  
64546 Mörfelden-Walldorf, Germany  
www.medintim.de

Tel. +49 (0) 6105 20 37 20  
Fax +49 (0) 6105 45 59 01  
Email: [service@medintim.de](mailto:service@medintim.de)

Made in Germany



0483 Medical device



[www.medintim.de](http://www.medintim.de)

  
**MEDintim**  
personal healthcare

## Less desire for sex – or impotent?

Many women themselves know that they can have less desire for sex. Some women have also experienced situations in which things didn't „work“ and they did not reach orgasm. Nonetheless, the intimacy was often enjoyable and the lack of orgasm was not a major problem. Perhaps it was also simply not discussed in the partnership.

This is different for men. Every woman soon notices, based on the erection and behavior of her partner, whether he has desire. And whether he can achieve orgasm. A man can scarcely hide impotence or feign orgasm.

In men, it soon becomes evident whether they are sexually fit or are having problems. Men who experience poor erectile function now and then are happy when their partner is understanding. However, if nothing works anymore – that is, no erection develops or his penis remains too flaccid for sex, problems arise.

## Feigning doesn't work

Some men gradually withdraw from shared sex. They change in the partnership, are often less affectionate and try to avoid sexual encounters. „Sleeping together every now and then“ turns into „It's not working together anymore“!

What women and also men are often not aware of is that initial and lasting impotence is often an early physical sign of a serious illness. It can be an indication early on of a heart attack. This is a physical sign! Your partner's erection problems are thus not due to a lack of interest in you or a love life that has become boring.

Often, after men reach the age of 50, impotence may be associated with cardiovascular problems. Diabetes and other illnesses as well as certain medications may be directly associated with erectile problems.

## How can I tell if my partner is having actual erection problems?

Try to find a calm moment to talk to your partner about this.

### As his partner, ask him exactly:

- ▶ How he is feeling,
- ▶ how long it has been since he had a normal erection?
- ▶ whether he is regularly taking medications, for example, for high blood pressure?
- ▶ whether he has ever had his blood sugar tested (diabetes)?
- ▶ whether he also otherwise feels worn-out and sluggish?
- ▶ when he last saw a urologist?
- ▶ whether he notices blood in his urine?
- ▶ whether he has pain in the area of the testicles and penis or in the pelvic area?
- ▶ whether he still sometimes has spontaneous nighttime erections? (If so, physical causes for erectile dysfunction are practically ruled out)

If your partner gives a positive answer to one of these questions, it is important to schedule an appointment soon with a GP he trusts or with a specialist (urologist)!

Important: In many countries, health insurance programs pay for annual medical check-ups starting at the age of 45. Unfortunately, too few men take advantage of this check-up with a urologist.

**We like to help you and your partner.  
Feel free to send us your personal questions!**

Contact: Martin Kessel  
Email: [kessel@medintim.de](mailto:kessel@medintim.de)



## Give your partner encouragement!

**„If only I had known earlier that my problem can be fixed so quickly and easily.“**

many of the men we have helped in the past 10 years tell us. It is unfortunate that men do not talk a lot about their problems with sex. Especially when it comes to impotence.

### Because nowadays, there are many aids and options:

#### ▶ Medications (Viagra® Cialis® Levitra® and other medications)

Note: These are not effective for every man and often have serious side effects!

#### ▶ Injection of a drug into the penile shaft to produce an erection (PSIT method)

Note: Not every man likes this method, since the drug must be self-injected into the penile shaft using a thin needle prior to sex. Also the glans frequently does not become erect, only the erectile tissue in the shaft.

#### ▶ Gel in the urethra (MUSE®): A gel which promotes erections and which is administered in the urethra beforehand.

Note: This method is not always effective. Introduction into the urethra must be practiced. Many men report burning during use.

#### ▶ Vacuum erection aids: An erection is created using a vacuum pump and maintained using a constriction ring.

Thirty minutes of sex under relaxed, stress-free conditions are possible. The vacuum pump is durable and reusable.

## Overview: Causes of erectile dysfunction and impotence

- ▶ **Metabolic diseases:** Diabetes mellitus, dyslipidemia, thyroid dysfunction, liver cirrhosis, kidney failure.
- ▶ **Cardiovascular diseases:** coronary heart disease, high blood pressure, arteriosclerosis, myocardial infarct.
- ▶ **Neurogenic diseases:** Stroke, Parkinson's disease, MS, herniated discs, paraplegia, nerve lesions due to surgery (prostate removal, rectal resection, bladder surgery)
- ▶ **Mental illnesses:** Depression, burnout or other mental illnesses.
- ▶ **Urogenital malformations and diseases:** Narrowing of the foreskin (phimosis), displacement of the urethral opening (hypospadias), tumor diseases, tear in the erectile tissue
- ▶ **Surgical procedures in the abdominal, pelvic and urogenital region:** radical prostatectomy, vascular surgery in the abdominal area, procedures on the prostate or testes, bladder and intestinal surgery.
- ▶ **Changes to the erectile tissue:** Penile curvature with hardening (Peyronie's disease, also known as IPP), fibrosis of the erectile tissue.
- ▶ **Drug side effects:** These include drugs for high blood pressure and heart disease (for example, beta blockers, ACE inhibitors), antidepressants and other drugs to treat mental illnesses (for example, sedatives, neuroleptics) as well as side effects of nicotine and alcohol abuse.