Animé Lubricant

Animé Lubricant mit Hyaluron

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Animé Lubricant

Animé lubricant with hyaluronic acid

Intimate lubricant for her and him

Instruction manual

Read the instructions for use before using.

Composition

Purified water, propylene glycol, glycerol, hydroxyethyl cellulose, benzoic acid, sodium hyaluronate

Indications

Gel to improve lubrication during sexual intercourse and nourish the upper layers of skin at the same time. Hyaluronic acid supports the healing of sore areas and smaller tears. For vaginal dryness. Can also be used on condoms and for inserting tampons or thermometers and aids for sexual stimulation.

Contraindications

Known hypersensitivity to the product. In cases of vaginal infection, ask your doctor before using the product.

Use

Apply the amount individually needed directly to the skin or apply to the desired site with a finger.

This product is compatible with condoms made of natural rubber latex, polyurethane and polyisoprene. It has **no** contraceptive effect. Suitable for vaginal use.

If the packaging is damaged such that the tube is no longer intact, do not use the product. Keep medical device out of the reach of children.

 Use within three months after opening.

Side effects

There are no reports of side effects to date.

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KESSEL medintim GmbH