



# ACTIVE LoopRing®

Adjustable constriction ring – re-usable

The **ACTIVE LoopRing®** is a medical device, used to treat men with erectile dysfunction or impotence. The adjustable constriction ring is intended to improve erections by increasing the fullness and firmness of the penis.

## Application:

The **ACTIVE LoopRing®** consists of an adjustable loop made of silicone rubber that is placed around the base of the penis and tightened. This pressure reduces the flow of blood from the penis to the body. The **ACTIVE LoopRing®** may be tightened or loosened to fit. An O-ring and ball-locking unit allow for easy adjustment of the tension. Reusable device.

The ALR could be used as well with the **ACTIVE Erection System<sup>NT</sup>** or **MANUAL Erection System®** (Vacuumtherapy), if the user wants to open the ring post climax and let sperm flow into the vagina.

## Material:

Loop: TPE (Thermoplastic Elastomere)  
Ring/Ball: ABS plastic

## Content:

1 **ACTIVE LoopRing®**  
Instruction for use  
Color: transparent  
Box: 65 x 93 x 19 mm

Art. Nr.: IA ALR  
EAN 40 13273 001922

Available in pharmacies, specialist retailers or directly from KESSEL MEDintim.



Medical Device

Made in Germany

