



**MEDintim**  
personal healthcare



# Prefert® Vaginal Gel

Lubricant when trying to get pregnant

## Application: women trying to conceive

Prefert® Vaginal Gel is a lubricant for couples trying to conceive at home. It is used prior to intercourse during the time of ovulation. During ovulation the properties of the cervix and cervical mucus are changing. This mucus is alkaline. Prefert® Vaginal Gel has a similar pH-value like cervical mucus during the fertile days of a woman (period of ovulation in the female cycle).

**When to use:** On the 3 most fertile days of the female cycle (time of ovulation).

## Mode of action

### Prefert® promotes sperm activity at the time of ovulation:

- Provides an optimal environment for sperm.
- Ideal pH for natural conception.
- Improves mobility of sperm and reduces the osmotic barrier to optimize the sperm cells migration.
- Prefert® Vaginal Gel improves osmolality.

## Properties:

- Odorless
- Tasteless
- Colorless (light opaque)
- Does not make spots
- Water-soluble
- Free of parabens
- pH-value: 7,0 – 7,4
- Osmolality: 315 ( 260 – 370) mOsmol / kg
- Viscosity: 2500 – 7000 mPa.s

**Ingredients:** Purified Water, Sodium Chloride, Sodium Phosphate, Potassium Phosphate, Arabinogalactan, Hydroxyethylcellulose, Polyhexanide. Paraben-free and silicone-free (dimeticone).

## Content:

- 4 pre-filled syringes Prefert® Vaginal Gel with 6 ml gel
- 1 vaginal applicator, re-usable

The content of this package is good for 4 applications (2–4 ovulation cycles). For home use.

Discard the empty syringes in household waste after use.

**Applicator:** Clean and rinse the vaginal applicator with warm water before re-use.

**Clinically tested in Germany.**

Art. No.: Gel Prefert

EAN 4013273001717

Made in Germany



[www.prefert.com](http://www.prefert.com)



KESSEL medintim GmbH  
Kelsterbacher Straße 28  
64546 Mörfelden-Walldorf, Germany

Tel. (06105) 20 37 20  
Fax (06105) 45 59 01  
E-Mail: [service@medintim.de](mailto:service@medintim.de)



[www.medintim.de](http://www.medintim.de)