



# Vagiwell® Dilators

Medical dilators made of silicone.

## Use:

The goal of the exercises is stretching (expanding) of the vagina.

The Vagiwell® dilator set is suitable for clinical and home use.

Starting with the smallest size, the dilator is introduced into the vagina and is kept there for several minutes. When one size can be introduced without any discomfort, move directly to the next size. The exercises should be performed in a relaxed sitting or lying position.

Unless prescribed otherwise by the physician, the exercises should be repeated daily. When the largest dilator can be inserted without any discomfort, sexual intercourse can be attempted.

Size	Color	Diameter A/B		Length C	
		mm	inch	mm	inch
Size 1	Light blue	12/14	0.47/0.55	122	4.80
Size 2	Mint	16/18	0.63/0.71	135	5.31
Size 3	Purple	20/22	0.79/0.87	150	5.91
Size 4	Pink	24/26	0.94/1.02	163	6.42
Size 5	Beige	28/30	1.10/1.18	177	6.97

## Indications:

- Dyspareunia of various etiology
- Vaginismus
- Scarring following episiotomy or perineal laceration
- During and after radiation therapy/ chemotherapy
- Following gynecological surgery with decrease in size of the vagina and scar formation
- Lichen sclerosus
- Vaginoplasty/ neovagina
- Tight vagina (virginity) for cultural or religious reasons
- Pain during gynecological examinations or when inserting a tampon

## Material:

Dilators: Silicone  
Pouch: Nylon

Description of the available sets on the back side.

CE Medical device





# Vagiwell® Dilators

## Available Sets:



## Vagiwell® Dilators Premium

5 dilators made of silicone:

Size	Color	Diameter A/B		Length C	
		mm	inch	mm	inch
Size 1	Light blue	12/14	0.47/0.55	122	4.80
Size 2	Mint green	16/18	0.63/0.71	135	5.31
Size 3	Purple	20/22	0.79/0.87	150	5.91
Size 4	Pink	24/26	0.94/1.02	163	6.42
Size 5	Beige	28/30	1.10/1.18	177	6.97

Art. Nr. Vagiwell P

EAN: 4013273001557




## Vagiwell® Dilators Small

3 dilators made of silicone:

Size	Color	Diameter A/B		Length C	
		mm	inch	mm	inch
Size 1	Light blue	12/14	0.47/0.55	122	4.80
Size 2	Mint green	16/18	0.63/0.71	135	5.31
Size 3	Purple	20/22	0.79/0.87	150	5.91

Art. Nr. Vagiwell S

EAN: 4013273001564

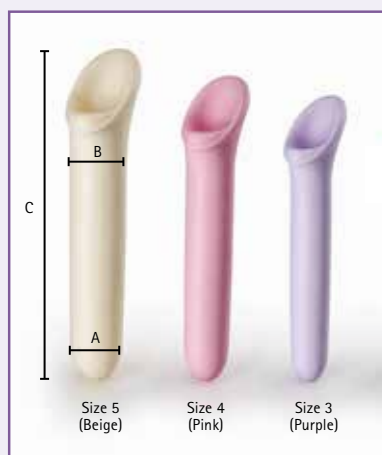

## Vagiwell® Dilators Large

3 dilators made of silicone:

Size	Color	Diameter A/B		Length C	
		mm	inch	mm	inch
Size 3	Purple	20/22	0.79/0.87	150	5.91
Size 4	Pink	24/26	0.94/1.02	163	6.42
Size 5	Beige	28/30	1.10/1.18	177	6.97

Art. Nr. Vagiwell L

EAN: 4013273001571

### Always included:

Instructions for use; SYLK® lubricant (water-soluble) 50 ml bottle;  
Storage pouch

### Material:

Dilators: Silicone

Pouch: Nylon

