

# Vagiwell<sup>®</sup> Dilators

Medical dilators for clinical or home use



Size	Color	Diameter A/B		Length C	
		mm	inch	mm	inch
Size 1	Light blue	12/14	0.47/0.55	122	4.80
Size 2	Mint	16/18	0.63/0.71	135	5.31
Size 3	Purple	20/22	0.79/0.87	150	5.91
Size 4	Pink	24/26	0.94/1.02	163	6.42
Size 5	Beige	28/30	1.10/1.18	177	6.97

Images displayed in original size.

CE Medical device

# Indications

In case of vaginal discomfort and pain e.g. during intercourse or medical examination due to:


- ▶ Dyspareunia (recurring or constant genital pain during sexual intercourse) of different causes
- ▶ Vaginismus (tightening of the vaginal muscles)
- ▶ scarring following surgical cuts to the perineum (episiotomy) or perineal tears
- ▶ during and after radiation therapy/chemotherapy
- ▶ subsequent to gynaecological operations (e.g. prolapse surgery or cancer operations)
- ▶ during and after radiotherapy/chemotherapy
- ▶ after vaginal surgery involving tightening of the vagina and scarring
- ▶ skin conditions such as certain lichen forms, e.g. lichen sclerosus
- ▶ vaginoplasty/neovagina (e.g. during sex reassignment surgery)
- ▶ narrowness of the vagina (virginity) described as a sexual impediment by those affected

Stretching exercises with the Vagiwell® dilators can also be useful in making gynaecological examinations or the insertion of tampons less painful.

## Application


- 1 Insert Vagiwell Dilator into the Vagina (start with the smallest size).
- 2 Leave the dilator in the vagina for a few minutes, then gently push and pull the dilator without removing it completely from the vagina.
- 3 Repeat this exercise 5 times, after that continue moving the dilator in a rotary motion.
- 4 Rotate the Dilator 5 times to the right and 5 times to the left. These movements help to gently dilate the vaginal tissue.

# Available Sets:

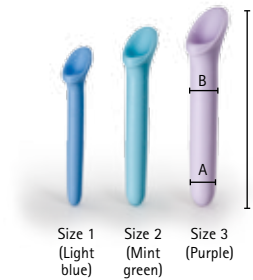


**Vagiwell® Dilators Small**  
3 silicone dilators sizes 1-3:

Art. Nr.: Vagiwell S  
EAN: 4013273001564

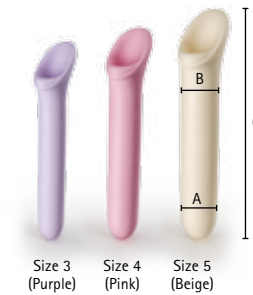

**Vagiwell® Dilators Large**  
3 silicone dilators sizes 3-5:

Art. Nr.: Vagiwell L  
EAN: 4013273001571

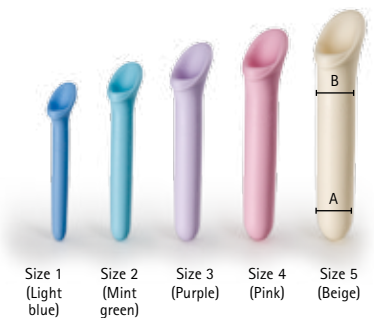

**Vagiwell® Dilators Premium**  
5 silicone dilators sizes 1-5:

Art. Nr.: Vagiwell P  
EAN: 4013273001557

Size	Color	Diameter A/B		Length C	
		mm	inch	mm	inch
Size 1	Light blue	12/14	0.47/0.55	122	4.80
Size 2	Mint	16/18	0.63/0.71	135	5.31
Size 3	Purple	20/22	0.79/0.87	150	5.91
Size 4	Pink	24/26	0.94/1.02	163	6.42
Size 5	Beige	28/30	1.10/1.18	177	6.97