Mode d'emploi:

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English

Instructions

Please read this information carefully before using the **AP Erektionring**. This device may cause injury if not used correctly.

Indications:

The **AP Erektionring** is a device used by men with light erectile dysfunction or impotence. The erection ring is intended to improve the quality of the erection of the penis.

What is Erectile Dysfunction (Impotence)?

Erectile dysfunction is commonly called impotence. Impotence is the inability of a man to get or to keep an erection firm enough to perform sexually. Normal erections are the result of increasing blood flow into the penis. This blood is then stored by temporarily closing off the veins that drain the penis. There are different levels of erectile dysfunction recognized. Some men cannot keep an

erection because the veins of the penis stay open and prevent enough blood from being stored in the penis. Many of these men can be helped with the use of the **AP Erektionring**. The **AP Erektionring** keeps the blood longer in the penis and the blood flow back to the body is reduced.

Diseases like diabetes mellitus, vascular disease, high blood pressure, certain nerve injuries, alcoholism, drugs and smoking may cause impotence. Impotence also may be a side effect of common medications. In those conditions, the **AP Erektionring** might not always be helpful.

Product Requirements

The **AP Erektionrings** are designed to support stiffness and shape of the penis. The **AP Erektionring** is made from silicone rubber and available in three sizes and easy to extend.

Application

The **AP Erektionring** is a device designed to improve erections by increasing the fullness

and firmness of the penis. The **AP Erektionring** is made of silicone rubber that is placed around the base of the penis behind the scrotum (testicles). This pressure reduces the flow of blood out of the penis. Instructions for use are below:

How to use the AP Erektionring:

1. Stretch the **AP Erektionring** before you have any arousal (stiffening). To choose the right rings size helps to achieve the best performance. Note to start with the biggest ring size.
2. At first: place the scrotum (testicles) through the ring, followed by the flat, non erected penis. The ring should be placed on the penis between the scrotum and the body.
3. When your arousal causes the erection (stiffening), the **AP Erektionring** reduces the blood flow back to the body. The erection will appear quicker, will be better and will remain longer.

Important to know

Talk to your healthcare provider if any of the following factors apply:

- If you have medical advices not to have sexual activities
 - If you have sickle cell anemia or trait, leukemia, tumor of the bone marrow or other conditions that change blood clotting and may result in bleeding or prolonged erections.
 - If you have an abnormally formed penis
4. The **AP Erektionring** makes a better performance and supports the naturally achieved erection. The positioning of the ring front of the pubic bone may stimulate the female partner at her clitoris.
 5. To loosen the tension: after intercourse or if the erection is too firm, the **AP Erektionring** may be loosened by pulling by hands on two ends of the ring. The firmness will reduce. Now the Erection ring can be removed by sliding out the penis, followed by the scrotum.

Warning and precautions

This product is made of silicone rubber and free of latex.

Do not use the AP Erektionring for more than 30 minutes per use. Use of the **AP Erektionring** for too long may cause permanent injury to the penis.

Care of the AP Erektionring:

if you use a lubricant for sexual activity, use only water-based lubricants. Mineral-oil based lubricants may decrease the life time and flexibility of the **AP Erektionring**.

Replacement:

repeated use of the **AP Erektionring** device may result in wearing out. In this case, replace the **AP Erektionring**.

Do not use the AP Erektionring if you do not have adequate use of your hands to easily remove the device.

Warning and precautions
This product is made of silicone rubber and free of latex.

Do not use the AP Erektionring for more than 30 minutes per use. Use of the **AP Erektionring** for too long may cause permanent injury to the penis.

Do not fall asleep wearing the AP Erektionring.

Call your doctor if a problem occurs. Stop using the **AP Erektionring** if these problems persist. Do not use alcohol or sedative when using the **AP Erektionring**. Do not use the **AP Erektionring** if you cannot sense pain in the penis.

Do not use the **AP Erektionring** if you do not have adequate use of your hands to easily remove the device.

Possible side effects

- Prolonged erection: if your erection is firm for more than 2 hours, call your healthcare provider immediately
- Permanent injury to the penis

if the **AP Erektionring** is either worn too tight or for more than 30 minutes.

Care of the AP Erektionring: if you use a lubricant for sexual activity, use only water-based lubricants. Mineral-oil based lubricants may decrease the life time and flexibility of the **AP Erektionring**.

The **AP Erektionring** is re-usable. Clean it after use with warm water and mild soap. Keep and store at a dry place.

Replacement: repeated use of the **AP Erektionring** device may result in wearing out. In this case, replace the **AP Erektionring**.

Do not use the **AP Erektionring** if you do not have adequate use of your hands to easily remove the device.

Possible side effects

- Prolonged erection: if your erection is firm for more than 2 hours, call your healthcare provider immediately
- Permanent injury to the penis