



Special Plus Extension

Large cylinder plus adapters for use with the **ACTIVE Erection System^{NT}**® or **MANUAL Erection System**®

Instructions for use:

The Special Plus cylinder is larger and longer than the normal standard cylinder.

Men, for whom the standard cylinder (dimensions: 200 mm length, 50 mm internal diameter) is too small and too short, can use this expander to help with their erections.

The Special Plus expander is not a stand-alone device and needs to be used in conjunction with the **ACTIVE Erection System^{NT}**® or **MANUAL Erection System**® pump heads and constriction rings. The Special Plus cylinder replaces the standard cylinder. The guide sleeves (grey) and the cone (grey) from the **ACTIVE Erection System^{NT}**® or **MANUAL Erection System**® are not used. The constriction ring is attached to the Special Plus cylinder using the cone.

Contents:

- 1 adapter for connecting the pump head (electric or manual) to the cylinder
- 1 adapter to the body
- 1 large cylinder:
length 250 mm, internal diameter 60 mm, external diameter 68 mm
- 1 cone for attaching the constriction rings
- 1 22 mm ring
- 1 tube of lubricant
- Information for use

Material:

Adapter: polyoxymethylene (POM)
Cylinder: transparent polycarbonate
Cone: polyoxymethylene (POM)
Constriction ring: thermoplastic elastomer

This product is a single-unit product and is suitable for extension of the **ACTIVE Erection System^{NT}**® or **MANUAL Erection System**®.

Art. No: AES SP
Made in Germany

