

THE SMART CONTRACEPTION


MEDintim
personal healthcare



Developed by
women for women!



Caya[®] contoured diaphragm

Diaphragm made of silicone rubber for contraceptive use

The **Caya[®]** contoured diaphragm is a new female contraceptive barrier device – easy-to-use and hormone-free. **Caya[®]** acts as a mechanical barrier that prevents sperms from entering the uterus.

Indications:

Women,
- who can't or don't want to use hormonal methods (e.g. due to increased risk of thrombosis)
- who can't or don't want to use an IUD
- who are breastfeeding
- who want contraception only when necessary
- after giving birth/ between two pregnancies
- who are allergic to latex
- in combination with natural methods of family planning (e.g. symptothermal method)

Properties:

- hormone-free
- easy to use – comparable to the use of a tampon
- safe and reliable
- no side effects
- economical: can be used for up to two years
- ecological: no waste, reusable
- latex-free
- single-size design fits broad range of women
- no impairment of sexual sensations of either partner
- only to be used when necessary

Use:

- is inserted into the vagina like a tampon before intercourse
- covers the cervix
- is placed between the back part of the vaginal dome and the space behind the pubic bone
- not to be used with lubricant based on silicon oil.

Contraindications

- in the first 6 weeks after childbirth
- when a conventional diaphragm of 60 mm and smaller or 85 mm and larger were previously used
- in case of strong descent of the pelvic floor/uterine prolapse
- not to be used with lubricant gel containing silicone (dimethicone)

Important

- it is recommended to always use **Caya[®]** in combination with a contraceptive gel (e.g. **Caya[®] Gel** EAN 4013273001250).
- **Caya[®]** does not protect from sexually transmitted diseases.
- **Caya[®]** diaphragm is available in pharmacies without a prescription in selected European countries.

Material

- medically approved silicone, nylon[®]

Contents

- one **Caya[®]** diaphragm
- detailed instruction for use
- storage box (case)

Cleaning

- with water and soap

EAN 4013273001854

 KESSEL medintim GmbH
Kelsterbacher Straße 28
64546 Moerf.-Walldorf
Germany

 0483
Medical Device
Made in Germany