# REHABI<sup>PVT</sup>®

## Peniler Vakuüm-Trainer

 Deutsch

 English

 Français

 Italiano

 Español

 Nederlands

 Dansk

 Suomi

 Norsk

 Svenska

 Polski

 Čeština

 Türkçe

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# REHABI<sup>PVT</sup>®

## Penile vacuum trainer

Medical device for penile erectile tissue training following prostatectomy and IPP (penile curvature).

### Important:

**Please read these instructions for use before using the erection aid.**

**Keep the instructions for use!**

### Instructions for Use

When the penis is stiff, this is called an erection. If a man has problems getting an erection, we speak of erectile dysfunction. The REHABI<sup>PVT</sup>® is a device that helps you to get your penis stiff. For this reason, the REHABI<sup>PVT</sup>® helps with erectile tissue training. This refers to a filling of the erectile tissue in the penis with blood.

### What are the causes of erectile dysfunction?

A variety of causes can lead to erectile dysfunction. The most common ones include diabetes, thyroid disease, liver or kidney disease, Treatment of hypertension, cardiovascular diseases. Prostate surgery often also leads to erectile dysfunction. Depression or exhaustion can also lead to erectile dysfunction. Sometimes erectile dysfunction is a side effect of medication, e.g. drugs against high blood pressure. Penile curvature, also known as IPP or Peyronie's disease, leads to problems during sex and possibly to impotence if the curvature is severe.

### The REHABI<sup>PVT</sup>® comprises the following parts:

- a pump head with a green handle [1]
- 1 transparent cylinder [2]
- a soft green ring loader [3]
- 1 tube of lubricant [4]
- 1 set of instructions
- 1 bag

### How to prepare the application?

**All product-related illustrations can be found on the cover flap of this booklet.**

- Push the pump head onto the larger opening of the cylinder. (Fig. 1)
- Attach the green ring loader to the other side of the cylinder. (Fig. 2)
- Apply some lubricant to the edge of the green ring loader. (Fig. 3)
- The REHABI<sup>PVT</sup>® is now prepared for use.

### How do I use the REHABI<sup>PVT</sup>®?

**The erection aid is exclusively for your personal use. It may not be passed on to third parties, nor be used by them.**

- Remove the pubic hair directly on the penis and around the root of the penis.
- The device must fit the skin tightly.
- Apply some lubricant to the edge of the ring loader.
- Introduce the penis through the ring loader into the cylinder.
- Press the device tightly to your body.
- Lift the green lever on the pump head. In this way, you will draw the air out of the cylinder.

### Do you see a good erection in the cylinder?

- Then allow the stiff penis to remain in the cylinder under negative pressure for two minutes.
- The erectile tissue is now filled well with blood.
- The green "air" button on the pump head is used to let air in.
- When you press the "air" button, the cylinder is bled (i.e. air is let in, reducing the vacuum). (Fig. 4)
- In this way you can easily release the erection aid from your body.
- Now you can repeat the procedure.

**Important: Performing erectile tissue training 1–2 times per day is good for penile circulation. Excessively frequent use does not automatically lead to more rapid success.**

### After use:

- Wait until the blood has flown back from the penis into the body.
- After using the REHABI<sup>PVT</sup>®, clean it as described in the section "How do I clean the REHABI<sup>PVT</sup>®?".
- Dry the parts.
- Put all parts back into the bag and keep the erection aid out of the reach of children.

### How does the REHABI<sup>PVT</sup>® work?

The pump head sucks the air around the penis out of the cylinder. This creates a negative pressure (vacuum). This causes the blood vessels in the penis to fill with blood, and the penis stiffens. As the cylinder is transparent, you can see whether the erection is hard enough. In most cases, pumping for 1 to 2 minutes will be sufficient to produce an erection.

### How do I know if the erectile tissue training is helping me?

Following prostate surgery, you can observe that continence is regained more quickly. This may mean several weeks of regular erectile tissue training. In the case of penile curvature (IPP or Peyronie's disease), the angle of the curvature may be gradually reduced. Experience has indicated that at least three months of regular erectile tissue training leads to an improvement in the condition. Some users have performed erectile tissue training for more than six months and achieved remarkable reductions in penile curvature. As a result of the improvement, it is possible that any planned surgery to correct penile curvature will no longer be necessary.

### Important information:

Never use the REHABI<sup>PVT</sup>® if you have drunk alcohol or taken drugs. Do not continue using the REHABI<sup>PVT</sup>® if it is no longer working properly. Also do not use the REHABI<sup>PVT</sup>® if you notice any damage to it.

In both cases, send the erection aid directly to the manufacturer **KESSEL medintim GmbH**. During the warranty period, repairs are free. If the device is not working or damaged because you have operated it incorrectly, you will always have to pay for the repairs.

If you do not live in Germany, please contact the dealer from whom you purchased the REHABI<sup>PVT</sup>®. Use only lubricants that can easily be washed off with water.

### Do you have any questions, or do you need help?

Feel free to call this number:

+49 6105 20 37 20

Or send an email to:

service@medintim.de

## Which side effects can the REHABI<sup>PVT</sup>® have?

### Do you suffer from cardiovascular disease?

Then talk to your doctor before you use the REHABI<sup>PVT</sup>® for the first time.

## The following side effects are possible:

You can get a bruise (haematoma) at the base or at the top of the penis.

Sometimes there may even be slight tissue injury, when the penis was not erect for a long time. However, that is not dangerous.

## When should I not use the REHABI<sup>PVT</sup>®?

Do not use the REHABI<sup>PVT</sup>® if you have one of these diseases:

- Diseases that affect the red blood cells, e.g. sickle cell anaemia
- Blood cancer (leukaemia)
- Bone marrow tumours
- Diseases that affect blood clotting (coagulation)
- Diseases that cause bleeding or prolonged erections
- Other diseases in which sex is not recommendable

Ask your doctor if you are not sure whether you can use the REHABI<sup>PVT</sup>® without any problems for your health.

## How do I clean the REHABI<sup>PVT</sup>®?

Remove the pump head from the REHABI<sup>PVT</sup>® before cleaning. All parts can be easily cleaned with warm water and mild soap. Do not use aggressive detergents. Clean the erection aid after each use. Store the erection aid in the included bag in order to avoid contamination.

## How do I properly dispose of the REHABI<sup>PVT</sup>®?

Dispose of the batteries into the special waste for

batteries. Dispose of the pump head into the special waste for small electrical appliances.

All other parts can be disposed of into the household waste.

## Warranty

Almost all parts are warranted for 2 years, as prescribed by applicable law.

Wear parts, such as constriction rings, are not covered by warranty. They must be replaced from time to time.

The warranty does not apply if you damage the REHABI<sup>PVT</sup>® or use it improperly.

You should send in the erection aid, if possible, every 3 years for technical inspection.

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