MEDintim

personal healthcare

# Caya<sup>®</sup> *contoured diaphragm*

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Diaphragma zur Kontrazeption

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**Instruction manual**

Contraceptive diaphragm

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**Mode d'emploi**

Diaphragme contraceptif

# Caya® *contoured diaphragm*

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
# Caya<sup>®</sup> *contoured diaphragm*

## Instruction manual

### Read the instructions for use before using the Caya<sup>®</sup> diaphragm.

Your Caya<sup>®</sup> diaphragm is a safe and easy-to-use contraceptive method. Always use Caya<sup>®</sup> in combination with a contraceptive gel (e. g. Caya<sup>®</sup> Gel). It is shaped to fit female anatomy. This shape and size were determined scientifically and fit most women.


### Your Caya<sup>®</sup> diaphragm looks like this: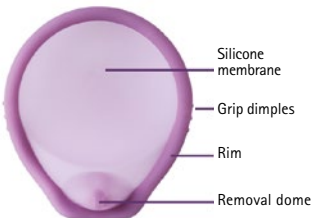

- **Silicone membrane:** covers the cervix to prevent sperm from reaching the uterus.
- **Grip dimples:** to facilitate vaginal insertion.
- **Flexible rim:** to ensure correct position.
- **Removal dome:** for removal out of the vagina.

### What is the position of the Caya<sup>®</sup> diaphragm in the vagina?

It completely covers the cervix in the upper vault of the vagina.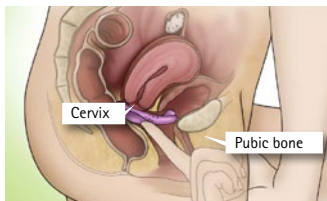

The Caya<sup>®</sup> diaphragm prevents sperm from reaching the uterus both during and after sexual intercourse.

**NOTE:** Please **always** use the diaphragm together with a contraceptive gel (e. g. Caya<sup>®</sup> Gel).


Before using the diaphragm for the first time, practise inserting it beforehand.

### Before using Caya<sup>®</sup> the first time:

- If possible, watch the animation on the internet at [www.caya.eu/en](http://www.caya.eu/en).

- Find the cervix – it feels like the tip of your nose.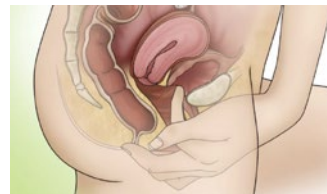

- Also find your pubic bone (in the vagina)

### How to insert the Caya<sup>®</sup> diaphragm:

**NOTE:** insert the Caya<sup>®</sup> diaphragm immediately before sexual intercourse.

1. Remove the Caya<sup>®</sup> diaphragm from the case. Before first use, the Caya<sup>®</sup> diaphragm should be washed with soap and water, rinsed and then dried with a soft clean cloth.
2. Apply the contraceptive gel, e.g. Caya<sup>®</sup> Gel, onto the middle of the membrane (approx. 4 ml).

Also applying some gel around the edges makes the diaphragm easier to insert.

Fold together using the grip dimples.

3. Find a comfortable position: lying down, kneeling down or with one leg elevated.

4. Hold the **Caya**® diaphragm with one hand. Place the thumb and index finger on the grip dimples along the rim. The arrow has to point towards your body.

5. Using your free hand, spread the labia. Push **Caya**® deeply along the posterior vaginal wall, towards your back, until the cervix rests inside the cup.

6. Use a finger to verify that the cervix is covered by the membrane of the **Caya**® diaphragm. The cervix must be completely covered. You should be able to feel it through the membrane. It feels similar to the tip of your nose.

If you cannot feel the cervix or the **Caya**®: Remove the diaphragm and try inserting it again.

**Caya**® is correctly positioned when the cervix is completely covered by the diaphragm and is between the posterior vaginal walls and the recess on the pubic bone.

If you have verified both of the above and the diaphragm is in the correct position, safe contraception is ensured. You are now ready for intercourse.

7. After intercourse, the **Caya**® diaphragm must remain in the vagina for at least 6 hours.

**NOTE:** The **Caya**® diaphragm should never be left in the vagina for longer than 24 hours without interruption, as wearing it for longer and uninterrupted periods of time can promote bacterial growth inside the vagina. These bacteria can then lead to Toxic Shock Syndrome (TSS).

In the event of repeated sexual intercourse, extra gel should be applied in front of the **Caya**® diaphragm without removing it beforehand.


8. To remove the diaphragm, insert one finger into the vagina and pull out the **Caya**® diaphragm using the removal dome.

#### 9. Instructions for cleaning and care

After use, the diaphragm should be carefully washed with soap and water, rinsed and then dried with a soft clean cloth. The diaphragm should not be heated in order to dry it. Synthetic detergents or organic solvents should not be used. Each time the **Caya**® diaphragm is used and washed, it should be examined carefully to ensure that there are no pin-holes or thin spots, the spring is fully enclosed by silicone rubber.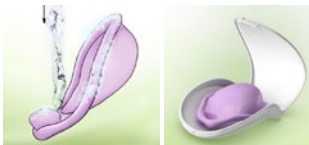

After treatment of a vaginal infection, the **Caya**® diaphragm should be replaced with a new one.

The **Caya**® diaphragm is reusable.

With appropriate care, it is expected the **Caya**® diaphragm should last for a period of two years of average use. If any indication of wear are observed, the diaphragm should be replaced immediately. Therefore, please note the date of first use on the case of the **Caya**® diaphragm. Silicone oil based lubricants have deleterious effect on products made of silicone rubber. Please use the recommended contraceptive gel or, if needed, a water based lubricant. The **Caya**® diaphragm should be stored in its case when not in use.

10. Check your **Caya**® diaphragm before you use it every time to make sure it is still ok.

When used in combination with a contraceptive gel, the **Caya**® diaphragm offers safe protection from unwanted pregnancy. However, no contraceptive method can guarantee 100% safety from unwanted pregnancy. **Caya**® does not offer protection against sexually transmitted infections.

**NOTE:** Always keep the **Caya**® diaphragm in the original case and out of the reach of children.

#### When using, never forget:

- Always use together with a contraceptive gel.
- Check position by feeling whether the cervix is covered by the **Caya**® diaphragm.
- After intercourse, leave in the vagina for at least 6 hours.
- Never leave in the vagina for longer than 24 hours without interruption.

#### Contraindication

- Use within the first 6 weeks after delivery/birth.
- If a conventional diaphragm with a size of 60 mm or 85 mm or larger has been previously used.
- In the event of severe pelvic floor prolapse/uterus prolapse.

#### Ask your healthcare provider, if:

- you have difficulty getting the **Caya**® diaphragm to fit in your body comfortably (rare).
- you use treatments for vaginal infections (rare)
- you have had lower abdominal/vaginal surgery (very rare).
- you have ever had a Toxic Shock Syndrome (very rare).
- you have any further, personal medical questions.

**NOTE:** If you have forgotten to use the **Caya**® diaphragm and have had unprotected sexual intercourse, please contact your doctor immediately to discuss emergency contraception.

#### Information

Tel:  
**+49 6105 20 37 20**  
**www.caya.eu**

#### Technical data

##### Material:

Medical silicone, nylon

##### Contents:

- One **Caya**® diaphragm
- Instruction manual
- Case for storage

##### **NOTE:**

**Never use the **Caya**® Diaphragm together with lubricant gels containing silicone. Intended for personal use only. Not to share with others.**

Due to production factors, there may be slight variations in the colour gradient. This has no effect on the quality and safety of the product.

## FAQs

### How safe is the Caya® diaphragm?

No contraceptive method can guarantee 100% safety from unwanted pregnancy. The **Caya®** diaphragm is just as safe as the classic diaphragm. The more often you use the **Caya®** diaphragm, the more confident you will become in using this contraceptive method.

### How do I know that Caya® is inserted correctly and fits?

Use your longest finger (usually the middle finger) to verify the correct position of the diaphragm: check whether the cup covers cervix and the removal dome is located securely behind your pubic bone. If this is the case you are well protected.

### For how long can I use the Caya® diaphragm?

After the first use, the **Caya®** diaphragm should not be used for a period longer than 2 years.

### Why does the Caya® diaphragm have to remain in the vagina for 6 hours after intercourse?

Sperm cells are no longer active after this period. These 6 hours are for your own safety and to prevent an unwanted pregnancy.

### Why should Caya® not be worn for longer than 24 hours without interruption?

Wearing the **Caya®** diaphragm for longer than 24 hours without interruption can promote bacterial growth inside the vagina. These bacteria could lead to Toxic Shock Syndrome (TSS).

### Can I have repeated consecutive intercourse with the Caya® diaphragm?

Yes, your **Caya®** remains in position, but you should apply extra contraceptive gel with the aid of an applicator to the area in front of the **Caya®**.

### Can the Caya® diaphragm also be used during menstruation?

The use of the diaphragm is possible.

### What do I have to observe in case of a vaginal infection?

Please visit your doctor for treatment in case of a vaginal infection (e.g. fungus). Please use condoms during treatment and NOT a diaphragm. After finishing treatment, you should replace the **Caya®** diaphragm with a new one.

### Can I use lubricants together with the Caya® diaphragm?

Yes, but ONLY water-soluble lubricant gels should, however, be used. Do not use ANY lubricant gels containing silicone together with the **Caya®** diaphragm!!! There are no studies on the tolerability of contraceptive gels when used together with commercial lubricants. Eventual limitations in terms of the efficacy of contraceptive gels cannot therefore be ruled out. Alternatively, you may use **Caya®** Gel as a lubricant. It is made up of lactic acid and cellulose, and is paraben-free. The **Caya®** diaphragm can be used with condoms that are coated with silicone oil.

### Could my partner feel the Caya® diaphragm?

This is highly unlikely. However, in rare cases, your partner may feel the rim of the diaphragm.

### Does the Caya® diaphragm influence my sexual desire?

Users report an improved libido compared to hormonal contraception.

### What should I do if Caya® causes pain during intercourse?

First verify the correct position of **Caya®** using a finger. If the **Caya®** diaphragm continues to cause pain, please visit your doctor.

### Can Caya® break or tear?

No case is known of a **Caya®** diaphragm becoming damaged during sexual intercourse.

### How can I positively influence safety when using the Caya® diaphragm?

The regular use and the use of a contraceptive gel (e.g. **Caya®** gel) are important measures for ensuring safety. In addition, the diaphragm should always be inserted before and not during intercourse.

### Can the Caya® diaphragm be boiled or disinfected?

After use, the **Caya®** diaphragm need only be cleaned with soap and water. Disinfectants do not need to be used and are not recommended. The **Caya®** diaphragm can, in principle, be boiled. After a vaginal infection, the **Caya®** diaphragm should be replaced with a new one.

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