



**MEDintim**  
personal healthcare



# Oralsafe

Latex sheet for oral-genital contacts

**Oralsafe** is a latex sheet to be used for oral-genital contact.

## Instruction to use:

Open carefully the foiled latex tissue. Moisture the vaginal or anal part of the body with a water-based lubricant. Place well the **Oralsafe** tissue to cover the vagina with clitoris or anus. Fix the tissue at its place while you have oral-genital contact. Dispose the used tissue at your waste. Don't flush into the toilette.

**Notice:** Use always a new **Oralsafe** sheet when you switch to use it from anal to vaginal areas. Do not cut or split up one sheet. Please use one sheet only one time. If you or your partner uses vaginally pharmaceuticals, please check with your healthcare provider if this pharmaceutical is compatible to latex. If you have latex-allergy, do not use this medical device – it may cause allergic reactions.

**Ingredients:**  
natural latex rubber

**Contents:**  
8 x 1 piece latex tissue for single use,  
available with strawberry or vanilla aroma  
size: 252 x 115 mm  
colour: beige  
box: 65 x 93 x 19 mm

**Storage:**  
Keep dry, cool and out of direct sunlight (UV).

Available in pharmacies, specialist retailers or directly from KESSEL MEDintim.

**Strawberry**  
EAN 4013273900164

**Vanilla**  
EAN 4013273900171



KESSEL medintim GmbH  
Nordendstr. 82 – 84  
64546 Moerfelden-Walldorf, Germany

Tel. +49 (0) 6105 20 37 20  
Fax (06105) 20 37 221  
Email: service@medintim.de

[www.medintim.de](http://www.medintim.de)