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*Applicator for single use (plastic)
for pessaries*

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Applicator for ovules
pessaries / rectal suppositories

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english

Applicator for ovules pessaries / rectal suppositories

Instruction for use:

Wash your hands and carefully dry them.

1. Carefully remove the pessary from the pack.
2. Insert the pessary into the applicator.
3. Insert the applicator nozzle as deeply as possible into the vagina. For rectal use, please place the suppository front of the anus, before you insert it with the plunger. This is most simply performed lying down with the legs slightly raised. In this position, the pelvic muscles are relaxed.
4. Carefully push the applicator plunger until it clicks, thus placing the pessary inside the vagina, or at rectal use into the anus.
5. Remove the applicator from the vagina or at rectal use: remove from the anus
6. Once used, dispose of single-use applicators with household waste.

Other important information for using vaginal pessaries

- Do not use pessary applicators during pregnancy. There is a risk of injury if handled incorrectly, as the area around the cervix is very sensitive in pregnant women. Instead, pessary treatment should preferably be performed without insertion aids. In general, the following applies: Ask your doctor before using pessaries during pregnancy.
- Do not administer treatment during menstruation. So that you do not have to interrupt your treatment, it should be completed before periods start. Talk to your doctor if you are shortly due to have a period at the start of treatment.

Note: pessaries or suppositories should not be larger than 14 mm width.

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