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Contragel green®

Instructions for use

(Please read carefully and keep for future reference.)

Contragel green® is a contraceptive gel for use with diaphragms and cervical caps.

Application

Before inserting into the vagina, place about one teaspoonful (about 4 ml) of Contragel green® in the diaphragm or cervical cap. Then follow the instructions for inserting the diaphragm or cervical cap.

The cellulose gel creates an additional mechanical barrier between the diaphragm (or a cervical cap) and the uterine orifice. The lactic acid with its physiological pH of 3.8 has a motility inhibiting effect on the sperms – they are slowed in their forward movement or stopped.

Composition

Water, lactic acid, sodium lactate, cellulose, sorbic acid, vanillin.

Storage

Store in a dry place between 4°C and 25°C. Always keep the tube closed. For minimum shelf-life, see printing on tube.

 Use within 3 months of opening.

Side effects

Contragel green®, because of its pH, is well tolerated by the vaginal flora. It can be used for as long and as often as required. If you notice any adverse effects during use, stop using Contragel green® and ask your doctor before further use.

Contraindications Contragel green®

There have been no reports of women experiencing side effects during the use of Contragel green®. Contragel green® should not be used if you are known to be hypersensitive or have an allergy to one of the ingredients. If you experience severe burning or itching during use, stop using the gel and contact your doctor.

Important

Contragel green® offers **NO** protection against sexually transmitted diseases. It is only effective when used together with a diaphragm (e.g. **Caya®** contoured diaphragm) or a cervical cap. Contragel green® **IS NOT** a safe contraceptive method when used alone without a mechanical barrier. Contragel green® is water-soluble and compatible with condoms made of natural rubber latex or polyurethane.

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KESSEL medintim GmbH