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Applicator for single use (plastic)  
for tubes with M9 thread  
(e.g. Contragel® green)

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HdB, Laagstraat 59, 9140 Temse  
Belgium

exclusive for distribution  
in The Benelux:

Memidis Pharma b.v.  
6546 BB, Nijmegen  
Nederland  
Tel.: +31 (0)246455411  
www.memidis.nl

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#### Instruction for use:

Thoroughly wash your hands.

1. Fill the applicator with the gel/cream. Attach the nozzle onto the tube and carefully squeeze the gel or cream out of the tube.
2. Remove the applicator from the tube.
3. Insert the applicator nozzle as deeply as possible into the vagina. This is most simply performed lying down with the legs slightly raised. In this position, the pelvic muscles are relaxed.
4. Once you have correctly positioned the applicator, empty it by gently pressing down on the plunger.
5. Remove the empty applicator from the vagina without pulling on the plunger, as some of the gel or cream would otherwise be sucked back into the nozzle.
6. Once used, dispose of single-use applicators with household waste.

#### Other important information for using vaginal cream and vaginal gel with the applicator

- Do not use cream/gel applicators during pregnancy. There is a risk of injury if handled incorrectly, as the area around the cervix is very sensitive in pregnant women. Ask your doctor for more information.
- Do not administer during menstruation. So that you do not have to interrupt your treatment, it should be completed before periods start. Talk to your doctor if you are shortly due to have a period at the start of treatment.

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