# SYLK

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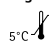
# SYLK

The natural lubricant from the juice of the kiwi plant has been proven for decades as a purely plant-based and thus well-tolerated lubricant. SyLK® is free of animal components, hormones, silicones and parabens.

## Instructions for use

Read the instructions for use prior to application.

## Composition

Purified water, sodium citrate, citric acid, potassium sorbate, grapefruit seed extract, vegetable glycerine, kiwifruit polysaccharide gum, xantha gum

## Indications

SyLK® is used to improve lubrication during sexual intercourse and possibly increase pleasurable sensations. As a result, fissures are avoided and infections occur less often. It is also suitable for use on condoms, for inserting tampons, thermometers and aids for sexual stimulation.

## Contraindications

Known hypersensitivity to the product. In cases of vaginal infection, consult your doctor before using the product.

## Directions

The amount individually needed is applied directly to the skin or to the desired site with a finger. Because of its mildly acid pH of 4.4 – 4.7, SyLK® should not be used when trying to conceive. In this phase, we recommend using Prefert® Vaginal Gel instead to optimize fertility.

This product is water-soluble, odorless and non-greasy and therefore it is compatible with condoms made of natural rubber latex, polyurethane and polyisoprene. It has no contraceptive effect. SyLK® is suitable for vaginal use.

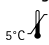
If the packaging is damaged such that the tube is no longer intact, do not use the product. Keep this medical device out of reach of children.

 Use within four months after opening.

## Side effects

To date there have been no reports of side effects.

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KESSEL medintim GmbH